

SafeBabyBMT Frequently Asked Questions

Is the SafeBaby System HL7 compliant?

Yes. SafeBabyBMT is able to receive any pertinent ADT information. We can also potentially import HL7 information back to the EMR(Electronic Medical Record).

How much does the SafeBabyBMT ADT interface cost?

The SafeBabyBMT interface is included with the basic SafeBabyBMT package.

Can SafeBabyBMT track the volume of Milk?

Yes. SafeBabyBMT has the capability to track mls from the moment the mother drops off the milk to the actual volume of milk a patient consumes. SafeBaby does not require a hospital to track volume, however.

Why does SafeBabyBMT use the mother as a primary contact when tracking EBM(Expressed Breast Milk)?

SafeBaby associates the milk to the mother. This allows the EBM to be more simply and securely used between multiple siblings. Once the milk is fortified or a syringe is filled, SafeBabyBMT tracks the milk off of the child's information.

Does the SafeBabyBMT system work with touchscreen devices?

Yes. SafeBabyBMT is compatible with any touchscreen monitor.

Can SafeBabyBMT use the hospital's Active Directory for login authentication?

Yes. The SafeBabyBMT system is designed to accept LDAP.

Does SafeBabyBMT require a hospital TO USE THE SafeBabyBMT Patient ID barcode for identification purposes?

No. SafeBabyBMT can use the patient's account number/MRN barcode on the existing ID Label to validate the patient.

Our hospital is not currently interested in tracking expiration and inventory with our EBM. Can we still use the SafeBabyBMT system?

Yes. The SafeBabyBMT system is designed to be as simple or as robust as your NICU requires. SafeBabyBMT can validate a feed in as little as 3 easy scans.

Does the SafeBabyBMT system accept and track donor milk?

Yes. The system can accept the milk and input the donor bank/lot number information. SafeBabyBMT can also assign donor milk to a specific patient.

Does SafeBabyBMT print up an Additive label after a bottle or syringe is fortified?

Yes. This feature has the ability to be turned on or off based upon a hospital's requirements.

What type of server does SafeBabyBMT run on?

Microsoft SQL.

Can we use our existing Server?

Yes.

Is SafeBabyBMT compatible with our wireless network?

SafeBabyBMT is compatible with any 802.11 network and all major wireless security protocols.

Can SafeBabyBMT be used with our existing Hardware?

Yes. The SafeBabyBMT application can be loaded on laptops, COW's, WOW's, and Monitors. It can also be loaded upon existing handhelds that use the Windows Mobile 5.0 OS or greater.

Does SafeBabyBMT track formulas.

Yes.

Does SafeBabyBMT track thawed and fortified EBM expiration parameters?

Yes.